

# BLUE-BLOKKER®

Light-curing blocking-out resin for orthodontics and telescope prosthetics

With excellent flow characteristics!



- Sealing saw cuts and blocking teeth and teeth segments after adjustment in the set-up technique.
- Precise application of the gel reservoir – (depending on the bleaching system) – in accurately defined thickness for bleaching splints and medical trays.
- Blocking out cervical margins, gaps, and strongly divergent teeth as well as fixing wire elements when fabricating hard elastic splints.
- Applying BLUE-BLOKKER® directly out of the dosing tip. BLUE-BLOKKER® can be very well combined with conventional waxes. Light-cured BLUE-BLOKKER® outer telescopes will be very stable and can be removed easily without tension. Even when working with higher thicknesses, BLUE-BLOKKER® will burn without residues. The results will show high dimensional stability and perfect fitting.
- The ideal viscosity and the micro and mini cannula allow precise application of the material. Curing with standard UV light in the spectral range of 380-470 nm. Depending on material quantity and type of light, BLUE-BLOKKER® cures within 10-30 seconds.



## Delivery Program

BLUE-BLOKKER® 4 × 1.5 g, incl. 10 cannulas	1 piece	5276
---	---------	------

In case you intend to duplicate the model using silicone, make sure to remove the inhibition layer of BLUE-BLOKKER® before duplicating. We recommend using for example a paper tissue soaked in ethanol.

