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EN Instructions for use

Intended purpose: Thermoplastic foils for the manufacture of appliances and splints for dental and orthodontic treatments.

Indication: Suitable only for the fabrication of aligner splints (for correcting tooth and jaw mis-alignments) and expansion aligners. Instructions for use can be found at www.scheu-dental.com/pressure-moulding-material.

Contraindication: Allergic reactions to components of CA® Foil (PETG) cannot be ruled out in sensitised people.

Any use not stated above is contraindicated.

Storage: Store at 5°C to 35°C. Protect from sunlight. Keep dry.

Important: Must only be used by qualified personnel. For single use. Not suitable for partial build-up with light-curing resins. Any serious incident involving the product must be reported to the manufacturer and the competent authority immediately.

Safety information: The practitioner is responsible for correct use. The splint must not be used if the material is cracked/broken/contains bubbles; there is a risk of swallowing or aspiration. When using CA® Foil with 3D-printed models, a non-perforated protective foil (ISOFOLAN®) or alternatively CA® Foil+ must be used. Remove the protective foil after the thermoforming process. Do not bring the products into contact with alcohol-based/organic solvents, hot water or steam jets.

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